GE Medical Systems

Direction 2174207

**LOGIQ 500/400
Advanced Vascular Calculations
Software Option**

H4550TY

This page intentionally left blank.

GE Medical Systems

Technical Publications

Direction 2174207

Revision 1

LOGIQ 500/400 Advanced Vasculature Calculation Software Option Installation Manual

Copyright © 1996, 1999 By General Electric Co.

Operating Documentation

GE Medical Systems

*GE Medical Systems: Telex 3797371
P.O. Box 414, Milwaukee, Wisconsin 53201 U.S.A.
(Asia, Pacific, Latin America, North America)*

*GE Medical Systems — Europe: Tel: +49 (0) 212 28 02 208
Beethovenstraße 239, Postfach 11 05 60, D-42655 Solingen
GERMANY*

Safety Warnings

WARNING

- This Service Manual is available in English only. If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services. Do not attempt to service the equipment unless this service manual has been consulted and is understood. Failure to heed this Warning may result in injury to the service provider, operator or patient from electric shock, mechanical or other hazards.

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Revision History

REV	DATE	REASON FOR CHANGE
0	November 1, 1996	Initial Release
1	October 21, 1999	Add Option On/Off Numbers

LIST OF EFFECTIVE PAGES

PAGE NUMBER	REVISION NUMBER	PAGE NUMBER	REVISION NUMBER
Cover Page			
Title Page	1		
Language Warnings i and ii	1		
Revision History A and B	1		
Procedure 1 thru 2	1		

Please verify you are using the latest revision of this document. Information pertaining to this document is maintained on GPC (Global Product Configuration). If you cannot access GPC, contact your Service Support Center.

This page left blank intentionally.

SECTION 1 – Installation

1-1 Introduction

LOGIQ™ 500/400 Software Options which include PDI (Power Doppler Imaging) Package, OB Calculations Package, Vascular Calculations Package, Advanced OB Calculations and Advanced Vascular Calculations are shipped from GE YMS turned off.

Shipping systems with these software packages turned off allows for tighter inventory control since we can deal with importing only one standard configuration.

Options NOT purchased by the customer should NOT be turned on.

1-2 Tools Required

NONE

1-3 Time Required

Approximately 30 minutes.

1-4 Parts/Documentation Required

1.	LOGIQ™ 500 User Documentation	
2.	LOGIQ™ 500 Service Manual	P9030TA
3.	LOGIQ™ 500 Proprietary Manual	P9030TB
4.	LOGIQ™ 400 User Documentation	
5.	LOGIQ™ 400 Service Manual	2127661
6.	LOGIQ™ 400 Proprietary Manual	2127662

1-5 Functional Check-out

Perform functional checks to verify that the system is functioning properly prior to installing any peripherals. Refer to the Functional Checks in Chapter 4 of the Service Manual.

1-6 Procedure

READ and UNDERSTAND these instructions thoroughly before proceeding with the installation. Perform each step in sequence and check it off when completed. If a problem occurs after completion, go back through the procedure and check for implementation errors before contacting your Ultrasound Region Engineer or the Online Center for assistance.

1. Ensure that all other installation procedures are completed and determine that the LOGIQ™ 500/400 is functioning properly prior to enabling (turning on) any authorized software options.

Turn on Advanced Vascular Calculations

2. Refer to the LOGIQ™ 500 Proprietary Manual (P9030TB) or LOGIQ™ 400 Proprietary Manual (2127662), section 2–4 for information on how to enter the service menu and select **03 Option Set**.
3. Refer to the LOGIQ™ 500 Proprietary Manual (P9030TB) or LOGIQ™ 400 Proprietary Manual (2127662), section 2–4–2 for information on how to install the software options purchased by the customer. For Advanced Vascular Calculations ensure that option number **12** is turned off and option number **26** is turned on.

Final Checkout

1. Check to ensure that all option packages installed are functional.
2. Refer to the LOGIQ™ 500 and LOGIQ™ 400 and ensure that the Advanced Vascular Calculations package functions normally.
3. When all functionality is verified, the software option installation is complete.